Nursing Anne (SimPad)

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Outline

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Session Viewer Open

SimCenter
Making Simulation Easier

Simulation Information / Event Log / BLS Result

Basic View

- 00:44 Obtain respiratory rate
- 00:45 Attach pulse oximeter
- 00:45 Obtain SpO2
- 01:13 Start ventilations
- 01:21 Set oxygen rate: 15 L
- 01:21 Oxygen device: Bag valve mask
- 01:21 Administer oxygen
- 01:21 兒童抽搐時，呼吸道通暢與呼吸支持是最重要的事。他的呼吸可能是淺快、不規則、且沒有效率的，最後可能會引發發紺。因此通暢呼吸道與提供氧氣，非常重要。
- 02:20 Prepare medication
- 02:22 Administer lorazepam
- 02:22 The appropriate first medication for seizures is benzodiazepines: diazepam, lorazepam or midazolam.
- 05:21 Administer diazepam
- 05:21 A second dose of benzodiazepine may be given, if seizure activity continues for 5 to 10 minutes after the initial dose. In this case it is less than 5 minutes since the first dose was given.

Back 00:00 85%

Sinus 80 120/80 (93) 12 37.2 98

Initial Airway Breathing Circulation

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SimPad編輯軟體_SimDesigner畫面

SimDesigner - Angina - Coronary Artery Disease_Chinese - A simulator-based training scenario for ALS Simulator (Changed)

Scenario Control Events Handlers Trends Flow chart view Settings Compatibility Monitor layout Help

SimDesigner LAERDAL

Patient data

 Name: Carl Shapiro
Age: 54 year(s)
Gender: Male
Height: 69 inch(es)
Weight: 110 kg
Picture
Include in learner brief

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☒ Show Learner brief on PM at startup
☒ Make Learner brief available on PM

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All other values are set to normal/default

Pneumothorax, left: OFF
CO2 exhalation: OFF
Pulse, femoral: NORMAL (Not supported)
Pneumothorax, right: OFF
Airway resistance, right: 0 %
Airway resistance, left: 0 %
Stomach distention: ENABLED (Not supported)
Pulse, neck: Normal
Pulse, right arm: NORMAL (Not supported)
Pulse, right hand: NORMAL (Not supported)
Chest rise: Normal

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Manual mode教案七組

 Acute Severe Asthma_English	SimDesigner document
 Acute_Severe_Asthma_English	Adobe Acrobat Document
 Angina - Coronary Artery Disease_Chinese	SimDesigner document
 Angina_Coronary_Heart_Disease_Chinese	Adobe Acrobat Document

Auto mode教案二組

Laerdal simulators compatible with Simpad system

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A. 325-00750 Breast Exam Module

B. 325-00650 Mastectomy Module

C. 325-00450 Fundus Skills and Assessment Module

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